

VSSI

O² Door




Instructions

Table of Contents

Parts Identification	2
Assembly	2-3
Features and Operation	3
Product Warranty	4

Parts Identification

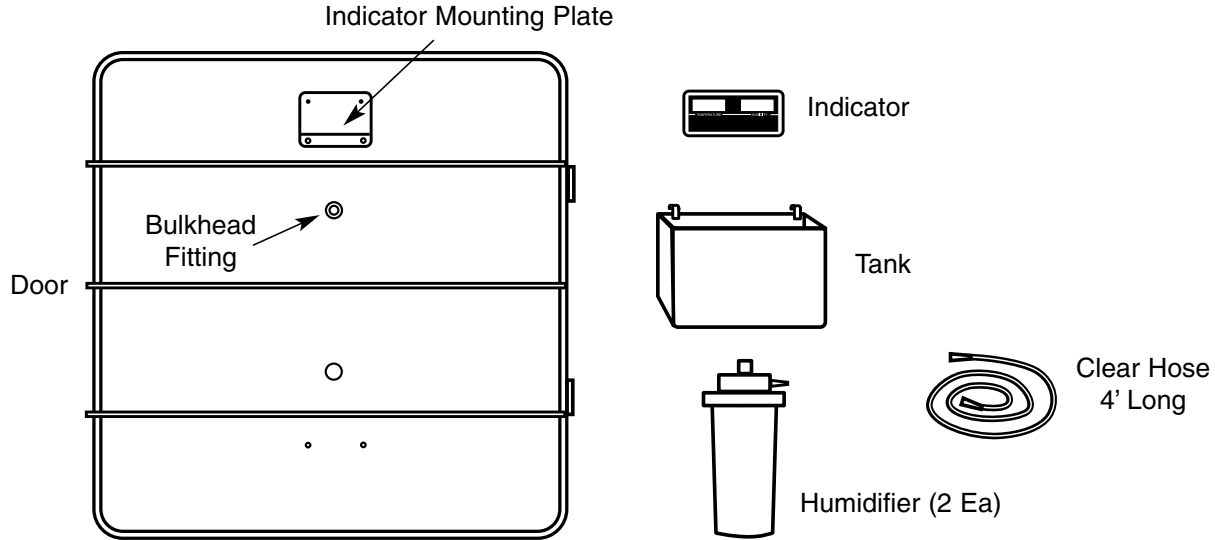


Assembly


1. Install door on cage by first removing existing cage door, and replacing with O² Door

NOTE: IF DOOR NEEDS TO BE REVERSED, SIMPLY REMOVE INDICATOR MOUNTING PLATE AND BULKHEAD FITTING AND REPLACE IN MOUNTING HOLES PROVIDED AT OPPOSITE END OF DOOR

2. Install tank by snapping tank hooks over horizontal cross member

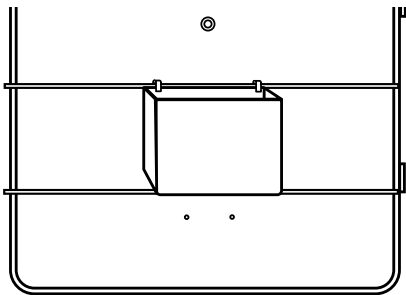


3. Place humidifier inside tank and lock into ring clamp

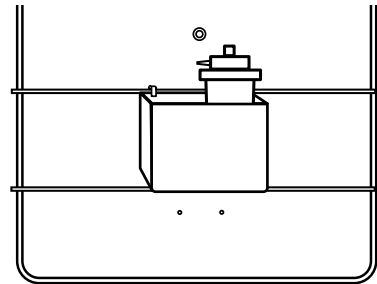

4. Remove clear hose from plastic bag. Use scissors to cut the 4' hose into two pieces at the desired length. The second piece can be stored for later use.


continued

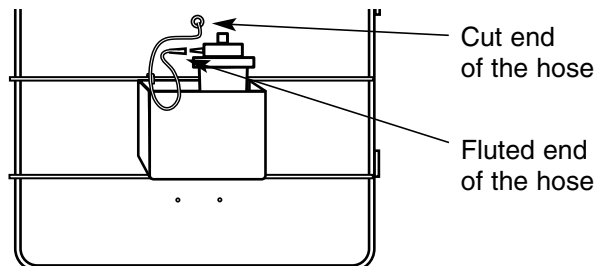

Assembly continued

5. Insert the fluted end of the hose on the similar shaped fitting on the humidifier. The cut end of the hose fits on the bulkhead fitting on the O² Door.
6. Install two AAA batteries (included) in the digital indicator. Refer to indicator instructions (packed with the indicator) for battery installation.

7. Insert the digital indicator into the mounting plate. Line up the screws in the plate with the holes in the back of the indicator. Once in place lock in place by pressing down on indicator.

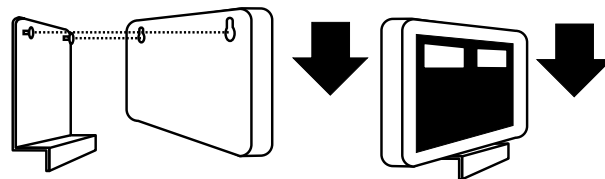

Features & Operation

1. Fill fluid container to line with appropriate solution.
DO NOT OVER FILL.
2. Depending on whether the oxygen needs to be warmed or cooled, place hot water, cold water, or ice inside the tank.
3. Connect fluid container to oxygen source.

NOTE: DOOR SEALS DIRECTLY AGAINST CAGE, THERE ARE NO SEALS TO INSTALL

4. Verify there are no leaks

WARNING: ALWAYS VERIFY OXYGEN FLOW AND PRESSURE SETTINGS. FAILURE TO DO SO COULD RESULT IN INJURY TO THE PATIENT.

Product Warranty

This product is warranted for 1 year from the date of delivery to the original purchaser only. Warranty is only applicable if customer complies with all instructions and specifications furnished by VSSI, Inc. relating to installation, care, and application. Customer agrees that they will not modify, misapply, or misuse product in any manner which deviates from the VSSI, Inc. instructions. Any repairs, alterations, or services provided by parties other than VSSI, Inc. or its authorized representatives may void warranty. The buyer is responsible for all freight charges on returned items.

Product Satisfaction Policy

In the event that the customer is not fully satisfied with the products purchased, VSSI, Inc. may, at its own discretion, arrange for the customer's account to be credited (excluding shipping charges) or replace the product. The customer must notify VSSI of any claim of nonconformity or defect in writing within 30 days from the date of receipt. The product must be returned to VSSI in its original packaging, undamaged and showing no unusual wear within 20 days of the customer's written notice.

To return an item, follow these simple instructions:

- Call VSSI, Inc. for complete instructions (800-299-9525)
- A sales representative will give you a Return Authorization Number
- If a Restocking Fee applies, you will be informed at this time
- Label the boxes with this Return Authorization Number
(All items must have a Return Authorization Number)
- Return the items, postage/freight prepaid

If necessary, the VSSI sales representative will assist you with freight arrangements.

Note: Custom orders and/or custom colors may not be returned.



#1 Civil War Road - Area 5 • P.O. Box 431 • Carthage, MO 64836
1-800-299-9525 • fax 417-358-4716
email vssi@vssi.com • www.vssi.com