


# VSSI

## *Premier Mobile Electric Lift Table*

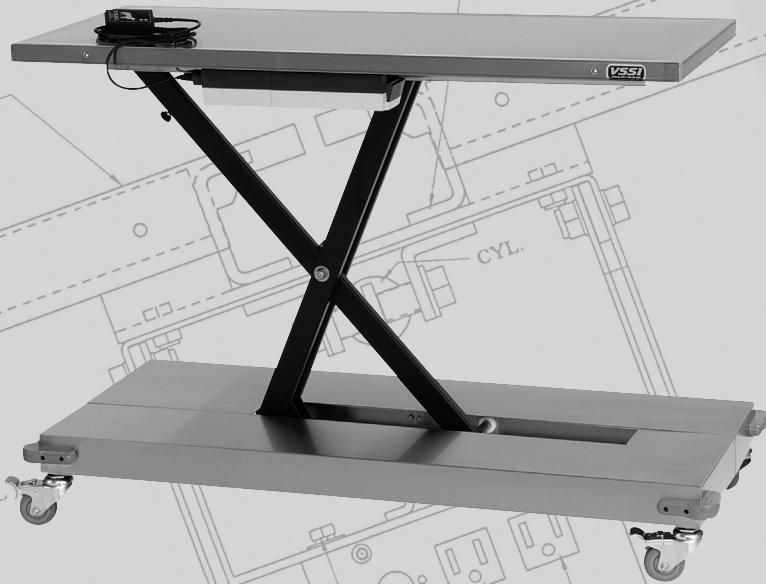

## *Features & Operation*

### *Table of Contents*

Installation .....	2
Charging the Batteries .....	2
Optional Scale Features/Operation .....	3
Troubleshooting .....	4
Care & Cleaning .....	4
Product Warranty .....	5

# *Installation*

---

Thank you for purchasing the Premier Mobile Electric Lift Table!

Open the package carefully. Please inspect shipping carton and contents for any visible damage. This is important in the event that any breakage has occurred during transit.

The Premier Mobile Electric Lift Table arrives pre-assembled, requiring only minor attention to be fully serviceable. After removing the table from its packaging, the table needs to be placed in the desired position within your facility. This is accomplished easily since the table is on casters. Use the simple hand control to raise or lower the height of the table.

It may be necessary to lift the Premier Mobile Electric Lift Table to remove it from the pallet. NEVER pick up the table by the stainless steel top! Lift by grasping the frame of the table base. Stainless steel

base covers are attached with Velcro® tabs, so be sure to reach far enough under edges of the base to be lifting by the frame, not the cover.

The Premier Mobile Electric Lift Table is protected during shipping with a white plastic film covering the stainless steel. Peel off the plastic film before using. Refer to “Care and Cleaning” instructions.



## *Charging the Batteries*

---

**Very important!**

**Please read carefully!**


The Premier Mobile Electric Lift Table utilizes rechargeable batteries to enable raising and lowering the table without having to be plugged into a power outlet at all times. Over draining these batteries will inhibit their ability to fully recharge, so it is very important to keep the batteries charged. Keep the table plugged in and in charging mode when not in use.

**To Charge Batteries:**

To charge batteries, plug the table into a standard power outlet. We recommend

charging the table every night, overnight, and more frequently if the table is being used continuously throughout the day without being plugged in.

The table needs to be unplugged for normal use.

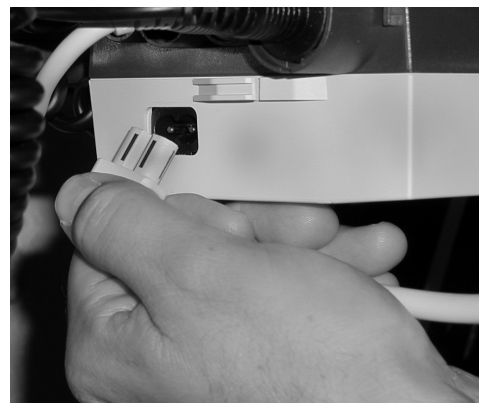

# Scale Features/Operation

---

## Using The Control Panel:



Push to turn unit on. Display shows the following:

- ^^^^ - self test
- vssi - checks display segments
- u 1.7 - software revision
- b xx.x - battery voltage  
(varies with AC plug in and internal 9v battery.) A reading of 6.0v indicates the need for battery replacement; the unit will shut off at a reading of 5.5v.
- 0.0 - ready to weigh



Push and hold until display indicates unit shift. Display will show “conv” until the conversion is complete.



Push the RECALL button - The display will show the message “rECL” then alternately flash between the last stable weight and the “rECL” message. Press ON/ZERO button to return to normal weighing. Works only with hold function active.

## Features:

### Tare Function:

Any weight within the scale capacity can be zeroed out by pressing the ON/ZERO button. When the weight is removed, the display flashes the tared weight. Press ON/ZERO button to return to normal weighing.

### Weighing Function:

Display will blank out until a close range of the weight is reached. With the hold function active, the display will show the stable weight and the “hold” indication. The displayed weight will lock until the weight is removed.

### Weight Hold:

Press and hold the ON/ZERO button. While holding the ON/ZERO button, press and hold the RECALL button. Display will show “zero” then “h on” or “h off”. Release buttons when selection is made. Default setting is “h on”.

### Automatic Shut Off:

In AC mode, the indicator will shut off in one hour. In battery mode, the indicator will shut off in one minute.

### Calibration:

Press and hold the ON/ZERO button. While holding the ON/ZERO button, press and hold the LB/KG button. After approximately 3 seconds the message on the display will change from “zero” to “CAL”. Release both buttons and the display will show “C0” message.

Make sure the platform is empty and press the ON/ZERO button. The indicator will show a count down from “16” to “1” then will show “C 50”. Place a 50 lb weight on the platform and press the ON/ZERO button. The indicator will show a count down from “16” to “1” then will show “50.0”. The indicator is now calibrated and has automatically returned to normal weight mode.

### Switching from AC to DC:

You must first unplug the AC adapter from the wall. Second, unplug the cord from the display. Then press the ON/ZERO button to turn the indicator back on.

### Switching from DC to AC:

Plug the AC adapter into a standard wall socket and the cord into the AC outlet on the back of the display. Then press the ON/ZERO button to turn the indicator back on.

# Troubleshooting

---

## Indicators:

Messages on the readout indicate operating condition:

OL Indicates weight on scale exceeds capacity or malfunction of scale display.

OPEN Indicates malfunctioning of base or cable, or the scale needs to be calibrated.



Indicates batteries are discharged.

## In The Event Of A Problem:

In the event that a problem occurs, a VSSI factory technician is ready to assist you. In order to save time, please check the following items before calling:

- Does the readout display have a message that indicates what the problem is? Refer to “Indicators” at left for descriptions.
- Check all cables/connections.
- Unplug AC power and battery, then plug-in again to reset the scale.

**VSSI Tech Support: 1-800-299-9525**

# Care & Cleaning

---

The Premier Mobile Electric Lift Table may be cleaned with any commercially available product recommended for use on stainless steel. Frequent cleaning will prolong the life and help maintain the natural luster of the stainless steel.

To prevent scratching, use a soft cloth to clean the stainless steel exam surface. If scrubbing is necessary, use a nylon or plastic type scrubber. When cleaning, always rub in the direction of the “grain” of the metal.

To remove minor scratches, we recommend using a 3M ScotchBrite® scouring pad rubbed lightly in the direction of the “grain” of the metal.

**Never use steel wool or steel brushes** when cleaning stainless steel. Steel particles may become embedded into the stainless steel and cause rust.

**Avoid use of chlorine bleach for cleaning.** Chlorides, bromides, iodides and thiocyanates will cause pitting, corrosion and discoloration of the metal. If using bleach, clean thoroughly with water to remove bleach residue.

**Avoid excessive moisture.** The Premier Mobile Electric Lift Table contains electronic components that may be damaged by excessive exposure to moisture. When cleaning the table or mopping the floor around the table, be careful not to splash water under the table base.

## ***Product Warranty***

---

This product is warranted for 1 year from the date of delivery to the original purchaser only. If equipped with the built-in VSSI Way™ Scale, Scale is warranted for 2 years. Warranty is only applicable if customer complies with all instructions and specifications furnished by VSSI, Inc. relating to installation, care, and application. Customer agrees that they will not modify, misapply, or misuse product in any manner which deviates from the VSSI, Inc. instructions. Any repairs, alterations, or services provided by parties other than VSSI, Inc. or its authorized representatives may void warranty. The buyer is responsible for all freight charges on returned items.

## ***Product Satisfaction Policy***

---

In the event that the customer is not fully satisfied with the products purchased, VSSI, Inc. may, at its own discretion, arrange for the customer's account to be credited (excluding shipping charges) or replace the product. The customer must notify VSSI of any claim of nonconformity or defect in writing within 30 days from the date of receipt. The product must be returned to VSSI in its original packaging, undamaged and showing no unusual wear within 20 days of the customer's written notice.

To return an item, follow these simple instructions:

- Call VSSI, Inc. for complete instructions (800-299-9525)
- A sales representative will give you a Return Authorization Number
- If a Restocking Fee applies, you will be informed at this time
- Label the boxes with this Return Authorization Number  
(All items must have a Return Authorization Number)
- Return the items, postage/freight prepaid

If necessary, the VSSI sales representative will assist you with freight arrangements.

Note: Custom orders and/or custom colors may not be returned.



#1 Civil War Road - Area 5 • P.O. Box 431 • Carthage, MO 64836  
1-800-299-9525 • fax 417-358-4716  
email [vssi@vssi.com](mailto:vssi@vssi.com) • [www.vssi.com](http://www.vssi.com)